

Recommended Use:

- Use 8-24 hours per day, for at least 2 hours per use.
Device may be used continuously.
- Device may be used while sleeping to receive therapy while you rest.
- Use the temperature control knob to choose a temperature that feels cool and soothing. If it is uncomfortable, turn up the temperature (CW = warmer), or discontinue use for 1-2 hours and try again. The water may be controlled between 50°F - 61°F (10°C -16°), so expect that it will not feel as cold as an ice pack.
- Begin with the temperature at the highest setting, and as you feel comfortable, reduce the temperature to increase cooling to your face.

Cautions:

- Check skin periodically. If a rash, irritation, or itchiness develops, discontinue use and contact your doctor.
- If the AQUECOOL™ unit leaks water, discontinue use and contact Aqueduct Medical at 877-365-HEAL.
 - *Note: Some condensation may occur on the Masque and around the quick disconnects. This is normal and it is OK to continue using the device.*
- Users who are sensitive to cold, or susceptible to injury from cold or compression (external pressure on the skin) should exercise caution when using the device.
- Extra care should be taken if the user has any of the following medical conditions: diabetes, incapacitation, impaired circulation and / or compromised skin.
- Extra care should be taken if the user is unconscious, under the influence of prescribed medication, undergoing extended therapy and / or has sensitive skin.

Troubleshooting:

If unit turns on, but does not seem to be cooling:

1. Make sure there is enough water in the system.
2. Ensure that all connectors are properly connected. Firmly press silver metal tabs until they "click" to release. Re-insert the connectors so that they both "click" into place.
3. If the unit is still not working, please refer to the more detailed troubleshooting section on page 19 of the AQUECOOL™ manual.
4. If you are still having problems, please contact Aqueduct Technical Service at 877-365-HEAL



AQUEDUCT MEDICAL, INC.

665 Third Street, Suite 20
San Francisco, California 94107

phone: (877) 365 HEAL
fax: (415) 543 2640
www.aqueductmedical.com

32-0012-002 Rev. B


AQUECOOL™ LOWER FACE & NECK Masque

Patient Instructions



Features:

AQUECOOL™ Device:

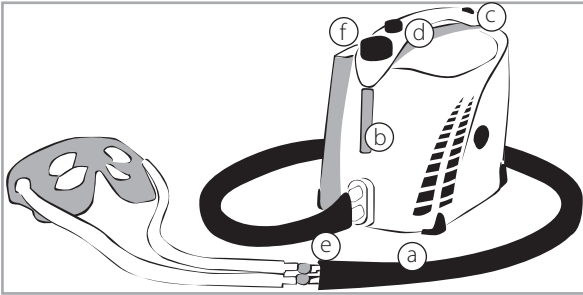

- a) **Device tubing:** (grey).
- b) **Water level window:** add water to MAX fill line.
- c) **On/off switch** Press firmly to turn device on or off. Green light will illuminate when unit is on.
- d) **Temperature control knob:** use control knob to adjust water temperature.
- e) **Quick disconnects:** (white) press silver tabs to quickly disconnect Masque. Turn off the AQUECOOL™ device before disconnecting Masque. To reconnect, insert the connectors until each one clicks.
- f) **Reservoir fill cap:** use distilled or filtered water.

AQUECOOL™ LOWER FACE & NECK Masque:




- g) **Adjustable head strap:** use the Velcro to adjust compression to the face.
- h) **Ear holes:** fit the ears in the holes and adjust the masque to apply compression to any swollen areas.
- i) **Compression strap:** angle strap to apply compression to swollen areas behind ear.
- j) **Cheek splints:** adjust splint so that Masque forms to cheeks.
- k) **Adjustable neckstrap:** use to support weight of tubing.

To Set-up the AQUECOOL™ Unit:


1. Fill the water reservoir to the max level.

- Use distilled or filtered water.
- Do not overfill.

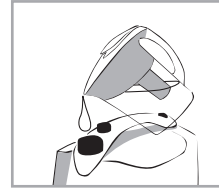


2. Connect the grey device tubing to the AQUECOOL™ unit.

- Insert Device tubing quick disconnects (male end) into AQUECOOL™ unit until you hear each one click.
- If you have difficulty connecting the tubes, firmly press silver tabs on the female end of the connectors until each one clicks. Then re-insert male connectors.

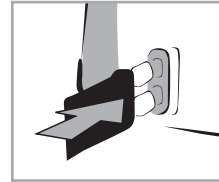

3. Plug the power cord into an electrical outlet.

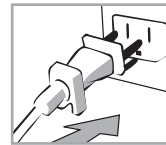


To Use the Masque:

1. Place Masque under chin and on face.

(Neck strap can be worn to support the weight of the tubing).



- Attach Velcro on top of head.
- Adjust for comfort and gentle compression.

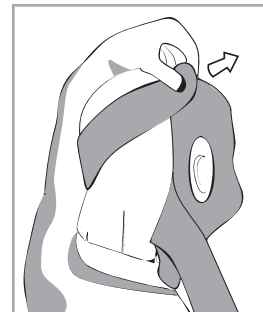

- Cross each compression strap behind head.
- Attach Velcro above ears.


To Use the Masque (continued)



- Adjust cheek splints so Masque forms to cheeks and temples.

2. Connect the Masque to the grey device tubing.

- Insert white connectors until you hear each one click (you should hear 2 clicks, one for each tube).

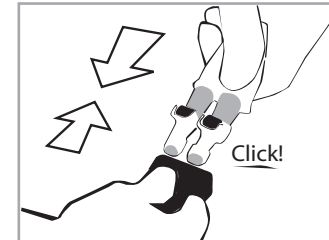

3. Turn on the AQUECOOL™ unit and adjust temperature as desired

- Check water level.
- Adjust temperature with temperature control knob.

4. To disconnect Masque, firmly press silver metal tabs on connectors until you hear each one click.

